



## Interpath Laboratory, Inc. Test File Update

---

As an Interpath customer who receives electronic results or sends electronic orders you may need to be notified when we update our Service Manual. Although we try to keep these changes to a minimum, laboratory medicine is an evolving industry requiring changes to our technology from time to time. Depending on the requirements of your EMR or Hospital Information System you may be required to make similar changes to your system in order to correctly process inbound electronic results and create outbound electronic orders.

If you are uncertain that you are required to update your system we recommend that you contact your vendor for more information. As your laboratory service provider we are available to participate in the discussion with your vendor so that you clearly understand the impact of these changes.

Included in this email:

- This cover letter with a summary of changes
- Microsoft Word® Document with the detail of these changes to our Service Manual
- Interpath Master Order/Result Compendium

Additional information including our most recent Service Manual and additional contact information can be found at [www.interpathlab.com](http://www.interpathlab.com)

**Effective Date: October 12, 2020**

---



# Interpath Laboratory, Inc.

## Test File Update

Order Code	Test Name	NC	CC	CPT	SRC	RRC	NT	DT	AOE
		Name Change	Component Change	CPT Change	Specimen Requirements Change	Reference Range Change	New Test	Discontinued Test	Ask on Order Entry Questions
76200	Heavy Metals Panel, Blood		◆				◆		

**76200 Heavy Metals Panel, Blood**
**CC/NT**

<b>Specimen:</b>	
Collect:	<b>One Royal Blue (EDTA)</b>
Submit:	<b>6 mL (Min:1 mL) Whole blood in Royal Blue (EDTA). Submit Ambient.</b>
Rejection Criteria:	Specimens collected/transported in tubes other than Royal Blue (EDTA) or Trace Element Free Transport Tube. Heparin anticoagulant or clotted specimens.
Stability:	Ambient: 1 Week(s); Refrigerated: 1 Week(s); Frozen: Unacceptable; Incubated: Unacceptable
<b>Methodology:</b>	Inductively Coupled Plasma Mass Spectrometry
<b>Performed:</b>	Mon-Fri
<b>Reported:</b>	2-8 Day(s)
<b>CPT Codes:</b>	82175 82300 83655 83825
<b>Interpretive Data:</b>	Please see report for interpretive data.
<b>Components:</b>	76057 - ARSENIC, BLOOD 76023 - CADMIUM, BLOOD 76029 - MERCURY, BLOOD <b>76041 - LEAD, BLOOD</b>

When updating panel please add component 76041 Lead, Blood and do not include 76043 Lead, Capillary from the previous update.